INSTRUCTIONS FOR USE

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MODE D'EMPLOI

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BRUGSANVISNING

EN ONLY USE SUREGRIP RAISERS UNDER STATIC FURNITURE. WEIGHT OF FURNITURE SHOULD BE EVENLY DISTRIBUTED. IF KNOCKED CHECK TO ENSURE STABILITY. CHECK FOR STABILITY BEFORE EACH USE. ALWAYS RAISE FURNITURE LEGS BY THE SAME HEIGHT. DO NOT USE ON UNEVEN SURFACES. TAKE CARE TO AVOID ENTRAPMENT OF FINGERS IN ADJUSTMENT SLOTS. **WARNING:** POTENTIAL TRIP HAZARD IF CROSS MEMBERS PROTRUDE OUTSIDE THE FOOTPRINT OF THE FURNITURE.

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EN If using “Raisers” on a hard floor fit the supplied three adhesive feet.

If furniture feet are 8.5cm or more use “Block Adapter” component, **follow A overleaf.** If furniture feet are less than 8.5cm, or have casters **follow B overleaf.**

To change height of “Raisers” place hand into base and turn to desired height. **Ensure all “Raisers” are the same height.**

If using “Link Arms + Spreader Bar”, first follow A or B, and then **C overleaf.** Ensure “Spreader Bar” is across the width of the furniture **not front to back - as this may cause a trip hazard.**

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VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

 [gordonellis.healthcare](#)
 [@EllisHealthcare](#)

LANGHAM SUREGRIP®

FURNITURE RAISERS

LINK ARMS + SPREADER BAR

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

SEE REVERSE FOR FITTING INSTRUCTIONS
AVAILABLE IN THE FOLLOWING LANGUAGES:
EN FR DE ES NL DK

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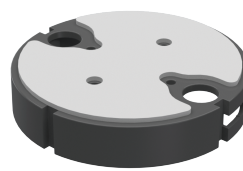
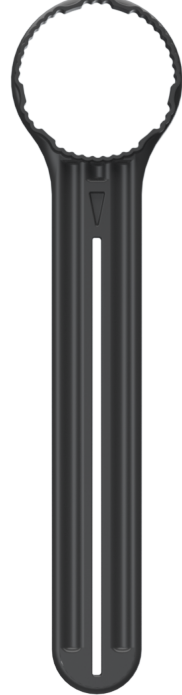
EC REP

MADE IN CHINA

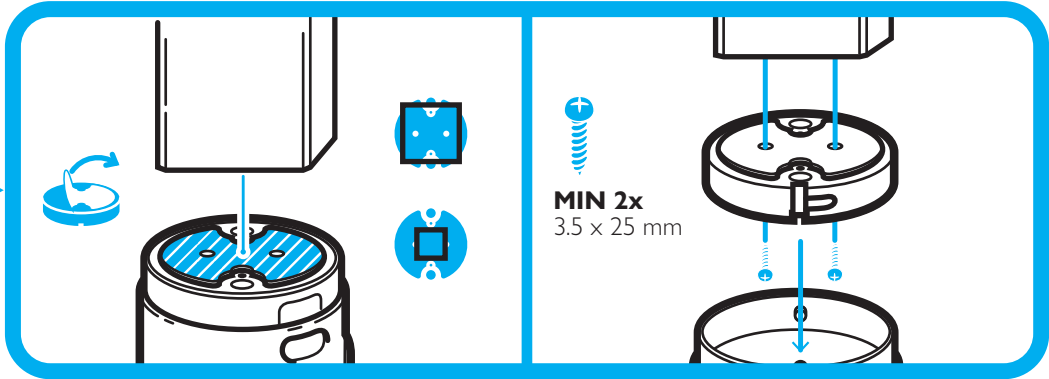
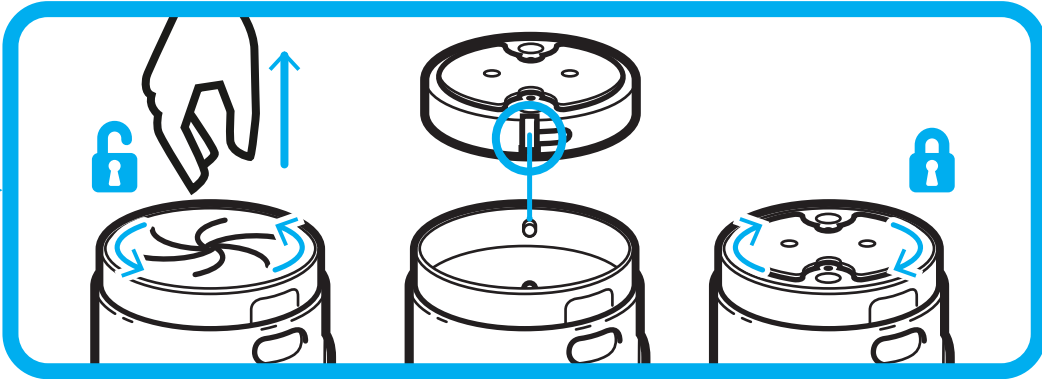
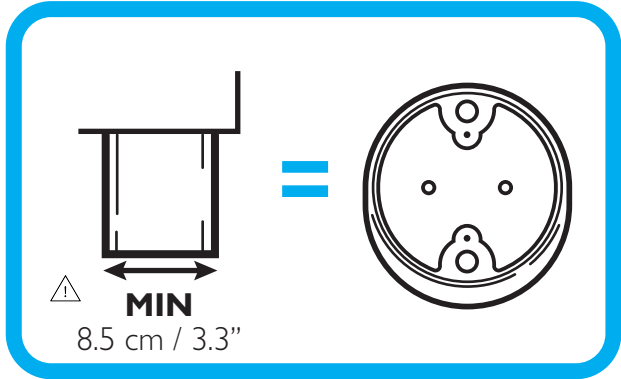
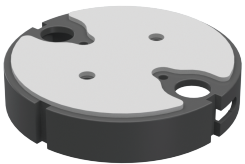
AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS CENTRE, 2ND
FLOOR, TOWER STREET, SWATAR, BKR 4013
MALTA

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK




 [gordonellis.healthcare](#)
 [@EllisHealthcare](#)





A BLOCK ADAPTER WITH ADHESIVE PAD

B FURNITURE RAISER 60570

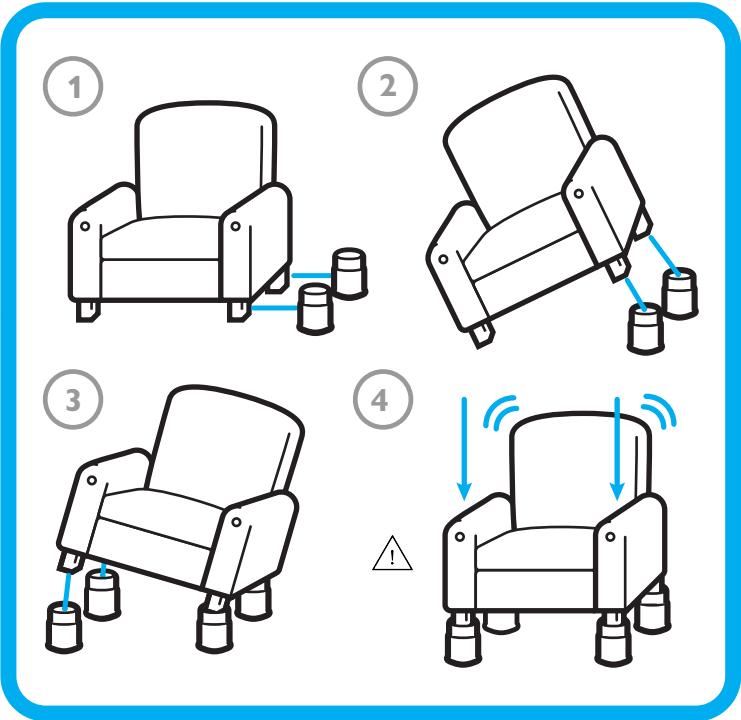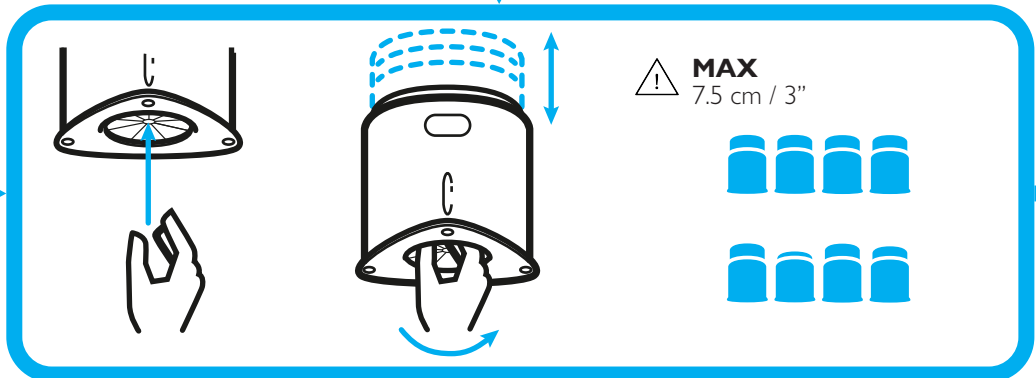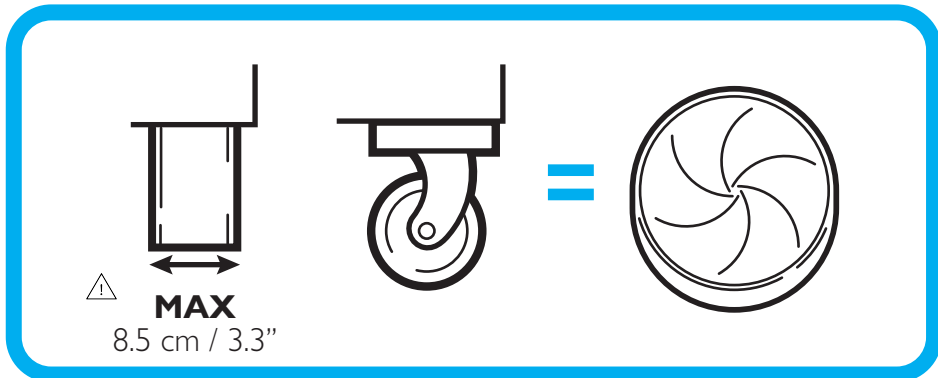C LINK ARM + SPREADER BAR 60571 + 60572

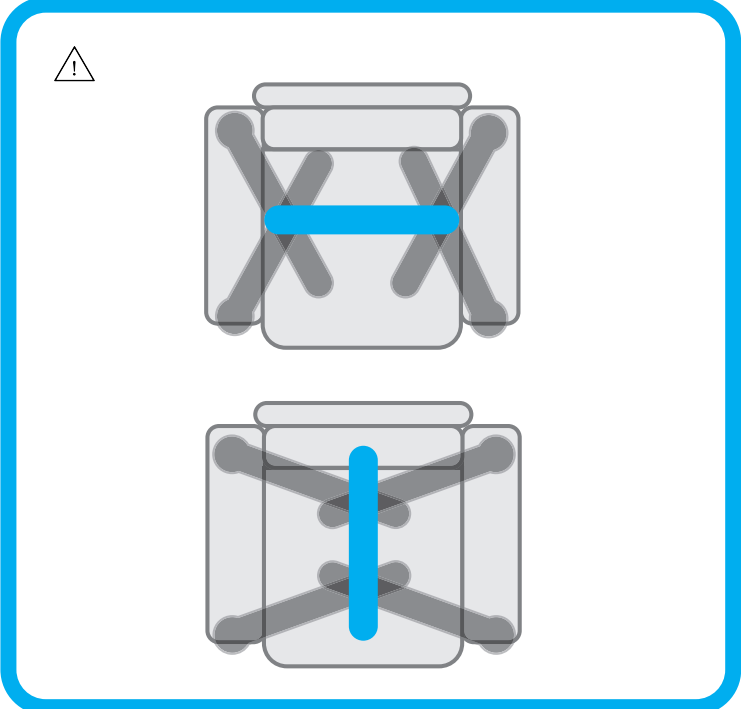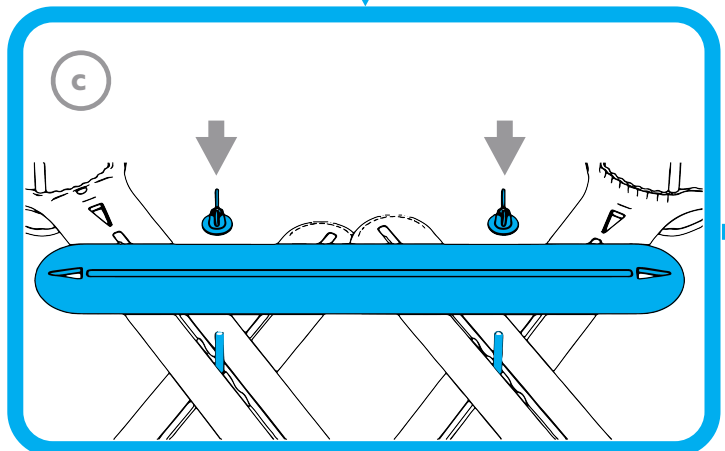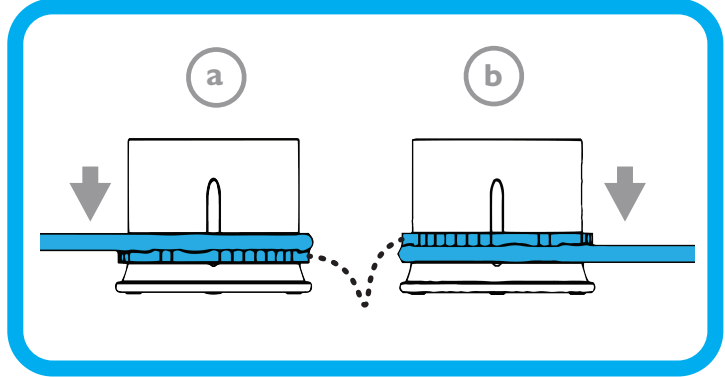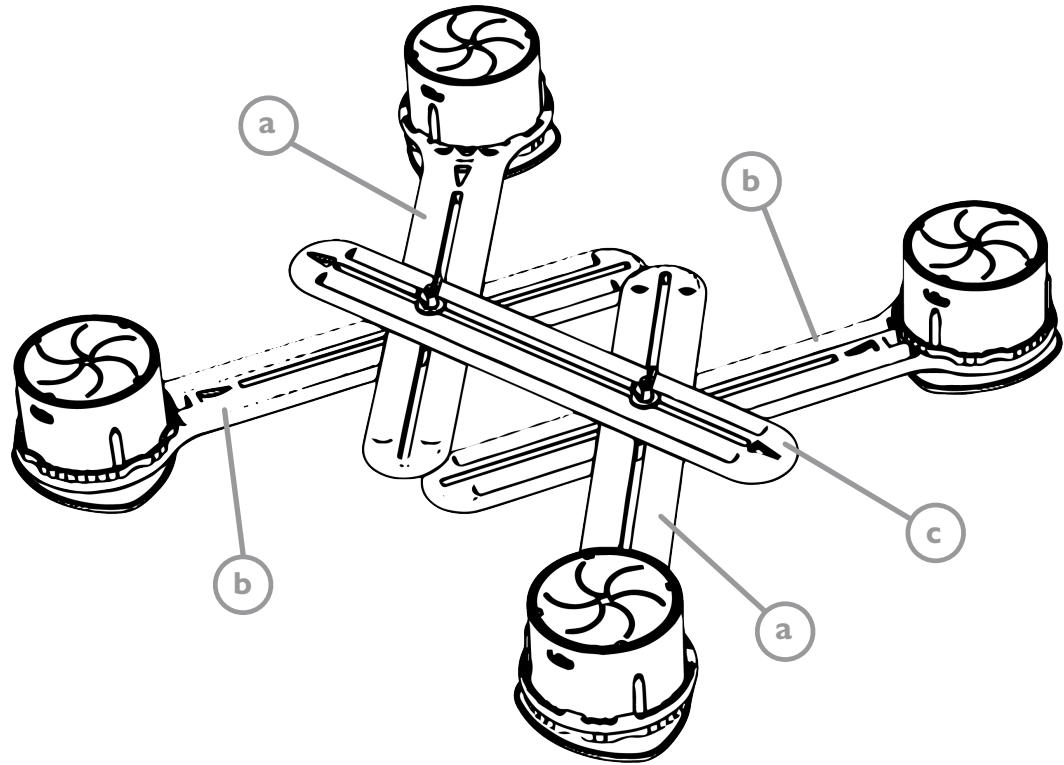FOR FURTHER INFORMATION AND GUIDANCE A VIDEO CAN BE VIEWED BY SCANNING THE QR CODE, OR VISITING:
POUR PLUS D'INFORMATIONS ET DE CONSEILS, UNE VIDÉO PEUT ÊTRE VISIONNÉE EN SCANNANT LE CODE QR OU EN VISITANT:
FÜR WEITERE INFORMATIONEN UND ANLEITUNGEN KANN EIN VIDEO DURCH SCANNEN DES QR-CODES ODER DURCH BESUCHEN VON:
PARA OBTENER MÁS INFORMACIÓN Y ORIENTACIÓN, PUEDE VER UN VIDEO ESCANEANDO EL CÓDIGO QR O VISITANDO:
VOOR MEER INFORMATIE EN BEGELEIDING KAN EEN VIDEO WORDEN BEKEKEN DOOR DE QR-CODE TE SCANNEN OF DOOR NAAR:
FOR YDERLIGERE INFORMATION OG VEJLEDNING KAN EN VIDEO SES VED AT SCANNE QR-KODEN ELLER BESØGE:

GORDONELLISHEALTHCARE.CO.UK/SUREGRIP-2