

Langham Plastic Half-Step LP/PHS




Maximum operating load



190-kg

As with all devices designed to assist the elderly or disabled, care should be taken to avoid sudden or undue movement, when using this equipment.

1. Position the Half-Step alongside the building entrance or threshold. The longest side of the step should be pushed up against the entrance or other static surface.
2. If necessary level the step by adjusting the feet using the supplied spanner (adjustment is provided for leveling of the step only, not height adjustment). This will enable a sound and level step to be set. Do not screw the feet out more than 38-mm measured from the underside of the step to the bottom of the feet (see diagram).
3. Check that the mat is correctly located on the step.
4. Tread centrally onto the step surface avoiding stepping onto the corners or along the edges of the step, and do not lean backwards as this is less stable for the user.
5. You may find it helpful to use a well-positioned grab to achieve a more secure transfer. This should be positioned at chest height on the outside wall opposite to the hinged side of the door. During transfer hold with both hands, place one foot first onto the centre of the step and then the other foot (see diagram) then step off.
6. The Half-Step can be cleaned using any all-purpose household cleaner.

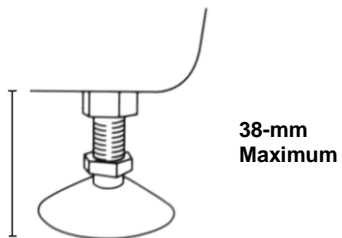

38-mm
Maximum

Langham
Products



TRENT LANE
CASTLE DONINGTON
DERBY DE74 2PY
ENGLAND
TEL: (01332) 850277
INT: + 44 1332 850277
FAX: (01332) 850366
INT: + 44 1332 850366



LPPHS/1110

