Welcome To Gordon Ellis & Co. Home Healthcare Products

**GORDON
ELLIS
& CO**

MANUFACTURERS IN
WOOD AND PLASTIC
ESTABLISHED 1893


**Prima
Multi Frame**


TRENT LANE
CASTLE DONINGTON
DERBY DE74 2NP
ENGLAND
TEL: (01332) 810504
INT: + 44 1332 810504
FAX: (01332) 850830
INT: + 44 1332 850830
gordon.ellis@gordonellis.co.uk
www.gordonellis.com

Company registered in England:
No. 231734
VAT No. GB 428 4666 26


65233/1205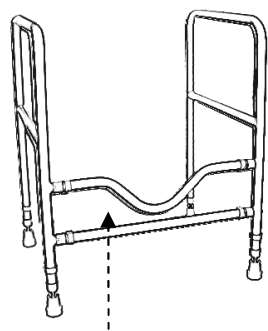

TOILET SURROUND –
Curved rail to front

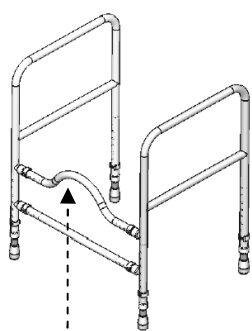TOILET CHAIR –
Curved rail to front of seat

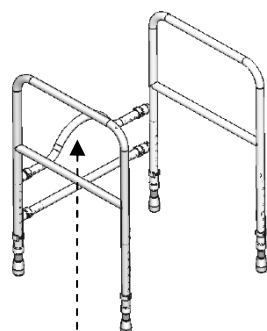SHOWER SEAT –
Curved rail to back of seat

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For other details of Gordon Ellis products for elderly and disabled people, please contact your supplier, or Gordon Ellis direct.

www.gordonellis.com

ENGLISH

Please keep these instructions for future reference. Refer to the assembly diagrams above to see how each model is assembled.

Intended Uses:

Can be used as a Toilet Surround, Toilet Frame or Shower Seat. Provides assistance in sitting down and rising from the seat.
Caution - This product contains dry natural rubber which may cause allergic reactions.

- The frame is designed to provide support and increase stability. It is not designed to support the total weight of an individual when used as a toilet surround only.
- When assembling the frame, decide upon the position of the curved rail by reference to the above assembly drawings.
- Ensure that all spring clips are fitted and protrude fully through the adjustment holes.
- Adjust the height of the frame by removing the spring clips and sliding the leg up or down to the desired height, and then replace the spring clips. **Warning:** Ensure that the spring clips protrude fully through the adjustment holes of the legs.
- Test for stability before use.
- Grip the hand rail securely and push down on the rail when raising or lowering yourself on to the seat.
- The product is designed for use in a static mode only. As with all devices designed to assist the elderly or disabled, care should be taken to avoid sudden or undue movement during use.
- Clean with mild, household cleaning products. Do not use abrasive powders or scouring pads. Do not use solvents to clean the frame.

OPTIONS

Floor Fixing Kit:

If it is required to secure the Toilet Surround to the floor, then remove the existing legs and replace with the Floor Fixing Legs. If in doubt consult a qualified tradesman.

Arm Rests:

Where supplied, fit the Arm Rests to the frame and screw together using the supplied screws. Refer to the instructions supplied with the arm rests.

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