Gordon Ellis & Co

Trent Lane
Castle Donington
Derby DE74 2AT
England

Tel: (01332) 810 504
Fax: (01332) 850 830
enquiries@gordonellis.co.uk
www.gordonellis.com

Company registered in England
No. 231734

VAT No. GB 428 4666 26

Part number:
6288/6289

 **Max Weight Limit:**
30 stone / 190kg / 420lbs

Made in China
Registered Design

Suitable for domestic
(household) use only.

6288/0217

Walton Commode

The Walton Commode is ideal for someone who has difficulty getting to their bathroom from their bed or other room in the home. This discreet weave design blends in with household furniture and features adjustable legs to make it the perfect height for the user.

Box Includes:

- 1 x Walton Commode with cassette, commode pan and lid
- 1 x Seat Pad
- 4 x Legs
- 4 x E-Clips

Assembly Instructions:


1. Insert legs into the bottom of the commode legs with the ferrule at the bottom (see diagram 1).
2. Push leg into the hole to the desired height and ensure that the holes in both legs line up.
3. Insert an e-clip into the hole at the desired height to secure the leg (see diagram 2). Repeat with all 4 legs ensuring the e-clip is fully inserted, with an audible click, into the holes in the leg and the commode (see diagram 3).
4. Check the legs are all positioned at the same height. Then test the commode for stability.

User Instructions:

1. Remove the seat pad and ensure the commode pan is correctly located in the cassette. Remove the lid from the commode pan before use.
2. Replace the commode pan lid, and using the metal handle transport the contents to a disposal point.
3. Clean the seating area and commode pan after each use with non-abrasive disinfectant.
4. Check the stability of the legs before each use.

Please Note:

- Ensure that weight is evenly distributed and feet are positioned firmly on an even floor when using the Walton Commode.
- Keep away from children unless supervised.
- Please discontinue use if you have any safety concerns when using this product, and contact us for advice.

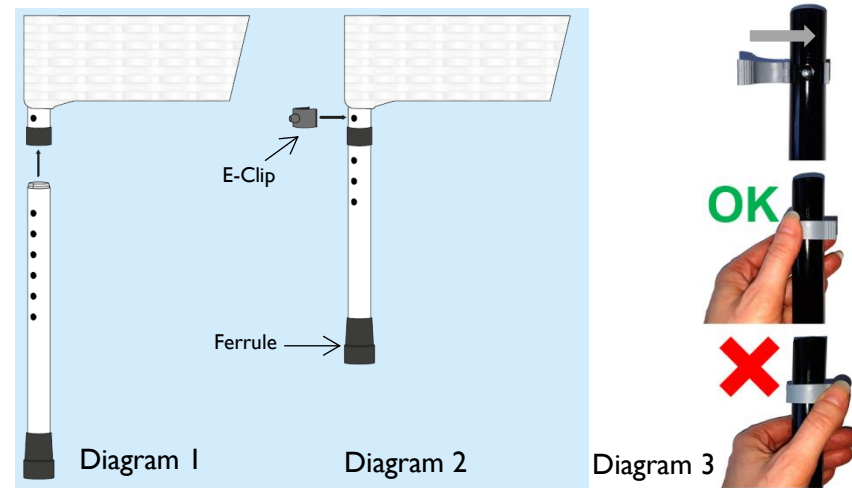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